





## Fitting Guide

### Preparation

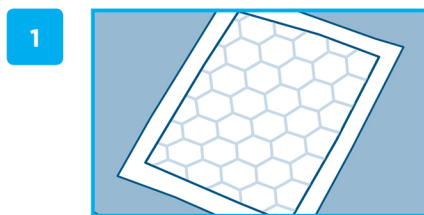


1 Open the product fully making sure that the waterproof back sheet is positioned on the bed and the soft white absorbent cushions are next to the skin.

### Fitting

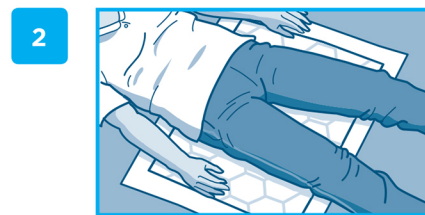


2 Position the person on the underpad.


### Changing

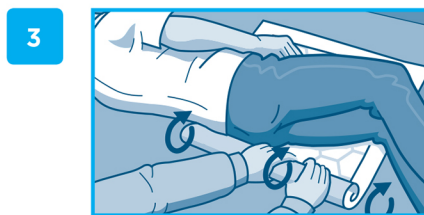


3 Turn the person into the lateral position and roll up the used underpad towards the persons back.

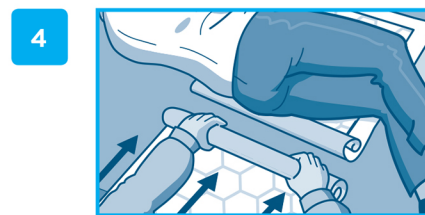

4 Unfold the new underpad, positioning it towards the back of the person.

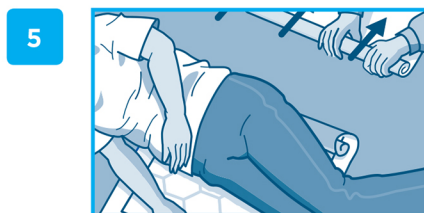

5 Turn the person on to the underpad and ensure that the pad is lying flat on the bed. Remove the used product.

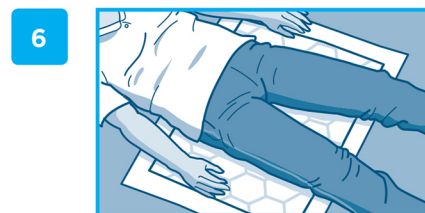

6 Lay the person in their original position.

## Hints and Tips

### Selection

- Choose the correct product size and absorbency dependent on the clinical situation you are using the product in.

### Leakage

- Check that you have the correct size and absorbency of underpad for the situation it is being used in.

### Disposal

- Roll up pads and dispose as per your Trust's clinical waste policy.
- Do not flush the pad down the toilet.



Attends Cover-Dri are tested and approved by proDERM, Institute for Applied Dermatological Research (does not include Cover-Dri Special Care).